

# Vario 18



## VERSATILITY AND EFFICIENCY



Vario 18 Suction Pump

# Vario 18

## Versatile and efficient suction

**Vario 18** is an efficient medical suction pump. An all-rounder for hospitals, clinics and homecare, it offers reliability, low noise and mobility. Its innovative QuatroFlex drive unit allows for easy servicing.

**Vario 18** can be used with either a Disposable or a Reusable Collection System, according to each customer's needs and demands.

Manufactured with swiss precision and quality, **Vario 18** offers a range of useful features:



- a Overflow protection/bacteria filter
- b Vacuum gauge in kPa and mmHg
- c LED warning light
- d Carrying handle
- e Membrane vacuum regulator
- f Tubing holder
- g Plug for 12VDC IN car connection cable
- h 1 L reusable collection system\*

\* also available with 1.5 L disposable collection system

### Unique swiss technology

The innovative 4-fold piston/cylinder system, QuatroFlex, builds up the vacuum within seconds. It also convinces through compact size and low noise during operation. As well as being unique in design and technology it is easy to service and maintain. A fact, highly appreciated by medical technicians. The membrane vacuum regulator allows for precise vacuum settings. It needs to be pressed down to change the vacuum, thus making it impossible to change the settings inadvertently. The Medela Vario 18 can therefore also be used at the patient's bedside.



Precise vacuum setting with press-down knob

### Versatility for a wide range of applications

Vario 18 is lightweight and portable for use across departments in hospitals and clinics and at home. It can be used for small surgical interventions as well as for airway suctioning. Variable suction levels can conveniently be set with the vacuum regulator. The Vario 18 can be equipped with either a Disposable or a Reuseable Collection System.



Optimal hygiene for many applications

### Hygienic and easy use

Vario 18 offers easy, intuitive handling. All accessories are designed to complement it perfectly and to prevent false connections. Optional accessories include a trolley, carrying bag and various attachments. It also features an inbuilt battery charger that allows for mobile use. The need to recharge is made audible and visible by special battery alarms.



Can be equipped for individual needs

#### Fast Facts

- Medela offers healthcare professionals 50 years of expertise in medical vacuum technology.
- Our swiss development and manufacturing process yields mechanical precision to guarantee excellent performance.
- QuatroFlex technology: silent function: can be used in environments in which quiet operation is necessary.
- Precise vacuum settings and fast vacuum build-up: ready when needed.
- One device for multiple needs.
- New users need little instruction to operate.
- Mains-independent for 30 minutes.
- Acoustic and optical battery management.
- Wide range of accessories available for individual application.
- Vario 18 incorporates reliability and ease of use like the Medela suction pumps for operating theatre.
- 2 years warranty on device.

Delivering excellence for over 50 years with  
comprehensive, high quality Swiss medical vacuum systems.

## Accessories

### Filters

to protect the pump from overflow, the environment from bacteria and to neutralise odour.



### Trolley

with rails for mobile transport of the pump and accessories.



### Reusable Collection System

autoclavable jars and lids collect secretions easily and economically. Sizes available: 1 and 2 litres.



### Holders and plates

can be used to attach Vario 18 to standard rails.



### Disposable Collection System

with disposable suction liners and reusable PC jars that offer a hygienic, simple handling and efficient fluid management. Sizes available: 1.5 and 2.5 litres.



### Carrying bag

for transporting the pump.



**Use Medela accessories for even greater flexibility and ease of use.**

### Plug for 12 VDC IN car connection cable

for operating the pump in the car.



## Technical data

	High vacuum -75kPa -563 mmHg
	low flow 18 l/min.

	AC 3,5 kg AC/DC 4,2 kg
	H x W x D 380 x 170 x 285 mm

	<b>IP21</b>
	ISO 9001 ISO 13485 CE (93/42/EEC), IIa
	2 years warranty
	<b>CE</b> 0123

CAUTION: Federal US law restricts this device to sale by or on the order of a physician.

## Medical Vacuum Technology for Healthcare Professionals

**Please contact us or your local Medela representative for details.**

 Medela AG  
Lättichstrasse 4b  
6341 Baar, Switzerland  
www.medela.com

### Local contact:

**Australia**  
Medela Pty Ltd.  
Medical Technology  
3 Arco Lane  
Heatherton, Vic, 3202  
Australia  
Phone +61 3 9552 8600  
Fax +61 3 9552 8699  
contact@medela.com.au  
www.medela.com.au

**Canada**  
Medela Inc.  
4160 Sladeview Cres., Unit #8  
Mississauga, ON  
L5L 0A1  
Canada  
Phone +01 905 608 7272  
Fax +01 905 608 8720  
info@medela.ca  
www.medela.ca

**India**  
Medela India private limited  
c/o Vatika Business Park  
First floor, tower 2,  
Sohna Road, Sec-49,  
Gurgaon 122 002  
India  
Phone +91 124 4416999  
Fax +91 124 4416990  
info@medela.in  
www.medela.in

**UK**  
Medela UK Ltd.  
Huntsman Drive  
Northbank Industrial Park  
Irlam, Manchester M44 5EG  
UK  
Phone +44 161 776 0400  
Fax +44 161 776 0444  
info@medela.co.uk  
www.medela.co.uk

**USA**  
Medela Inc.  
P.O. Box 660  
1101 Corporate Drive  
McHenry, IL 60050  
USA  
Phone +1 877 735 1626  
Fax +1 815 307 8942  
suction@medela.com  
www.medelasuction.com